

www.meko.de

MedTech Materials Webinars

POLYMER COATINGS FOR IMPLANTS

Learn how advanced polymer coatings provide tailored, biocompatible, and drug-releasing solutions – while seamless in-house processing fast-tracks your path to market.

OCTOBER 8, 2025

5 to 6 p.m. CEST, UTC+2

OCTOBER 9, 2025

9 to 10 a.m. CEST, UTC+2

MeKo
MEDTECH





PRECISION IS OUR PROFESSION

MeKo.de

MeKo
MEDTECH

Polymer Coatings for Implants

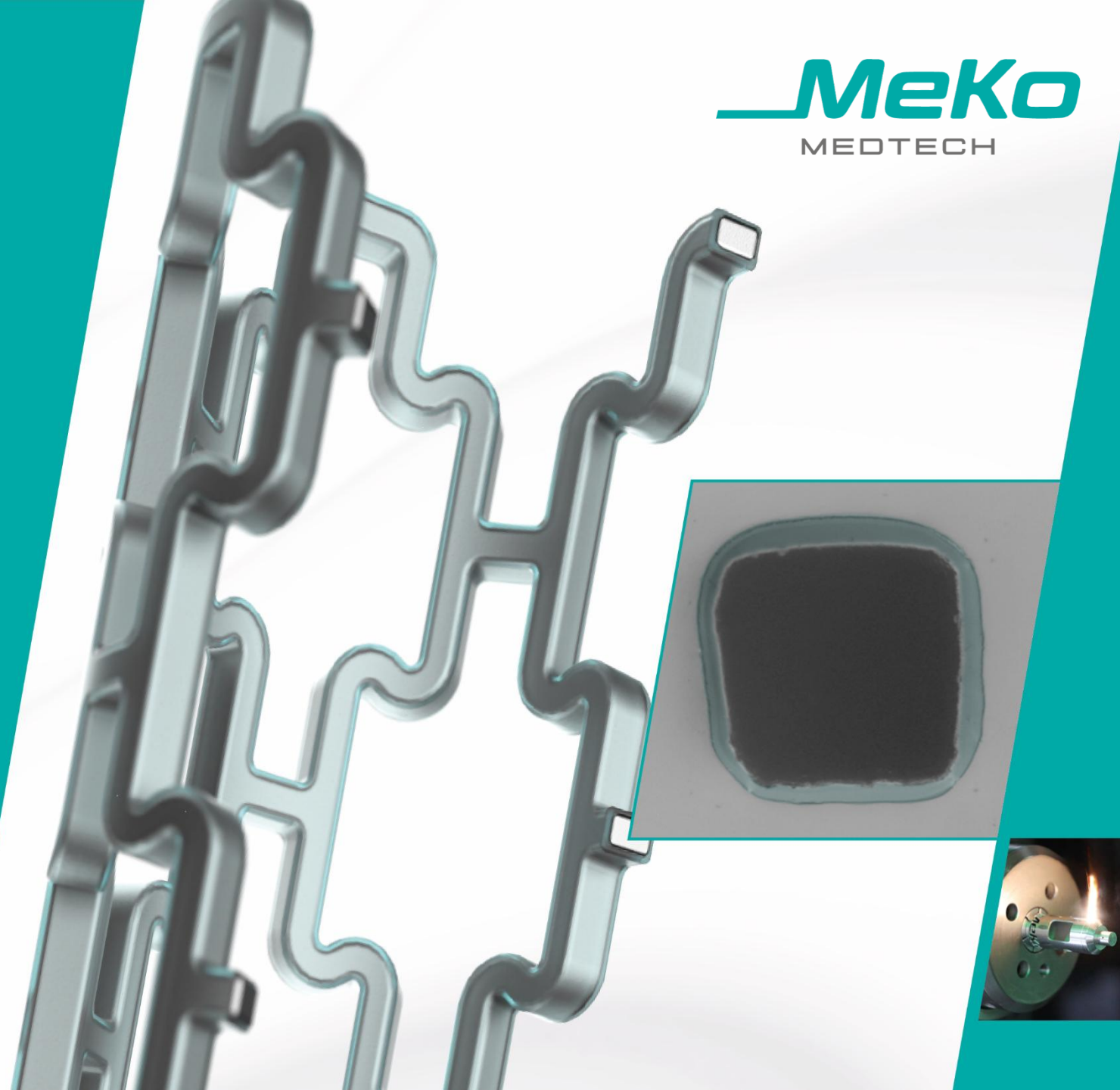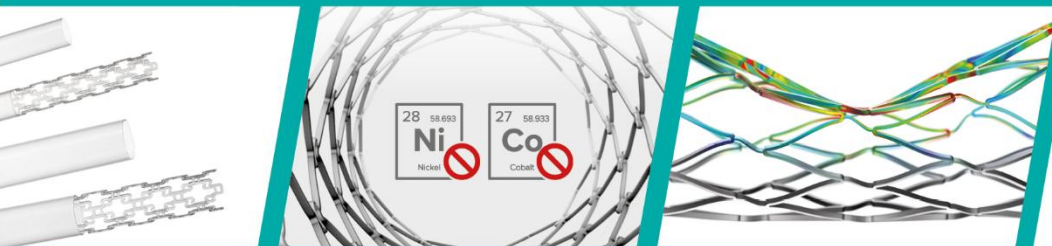

Agenda

MeKo MedTech

Polymer Coatings for Implants


Questions & Answers



Our History



In 1991 Clemens Meyer-Kobbe started with one laser in a garage near Hannover, Germany.

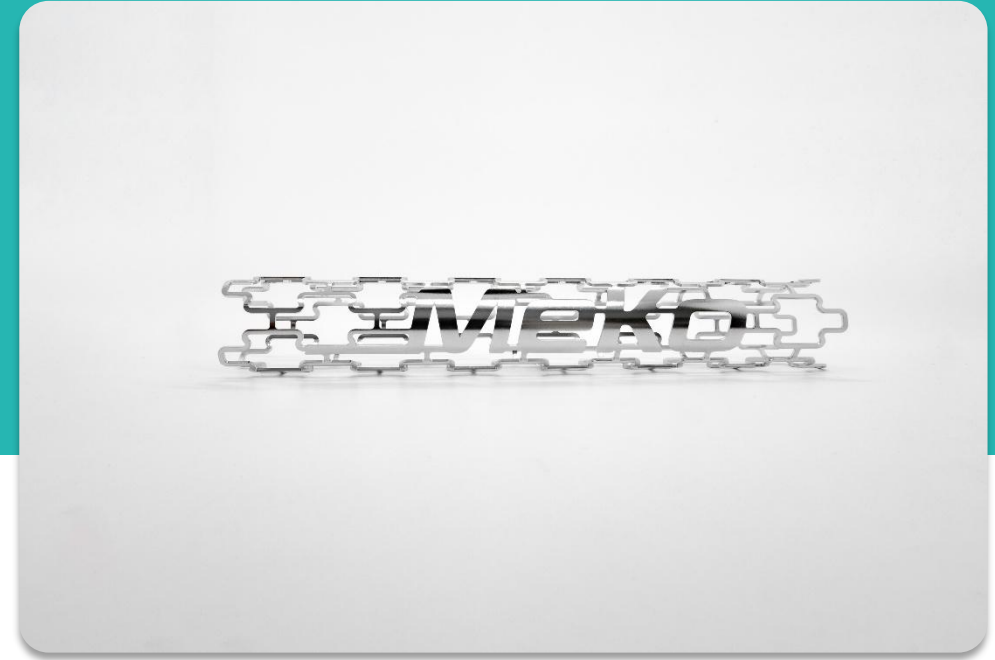

In 1995 MeKo was one of only two companies in the world laser cutting human-use stents.

Present Day

ISO 13485
ISO 9001

FDA
registered

160,000
sq. ft.



Working for the Future

Our dedicated team is constantly innovating and elevating standards.

Materials

Manufacturing Processes

Inspection & Analysis

Automation

Working for the Future

Our dedicated team is constantly innovating and elevating standards.

Materials

- **Resoloy®**: new magnesium alloy with **>30% break elongation** currently used in FIH-study
- **PolymediX®**: high-precision polymer tubes with **drug integration** into tube wall
- **Vasculoy®**: enhanced biocompatibility due to absence of Nickel and Cobalt in balloon expandable stents

Working for the Future

Our dedicated team is constantly innovating and elevating standards.

Manufacturing Processes



Working for the Future

Our dedicated team is constantly innovating and elevating standards.

Manufacturing Processes





PRECISION IS OUR PROFESSION

MeKo.de

MeKo
MEDTECH

Polymer Coatings

For Medical Devices

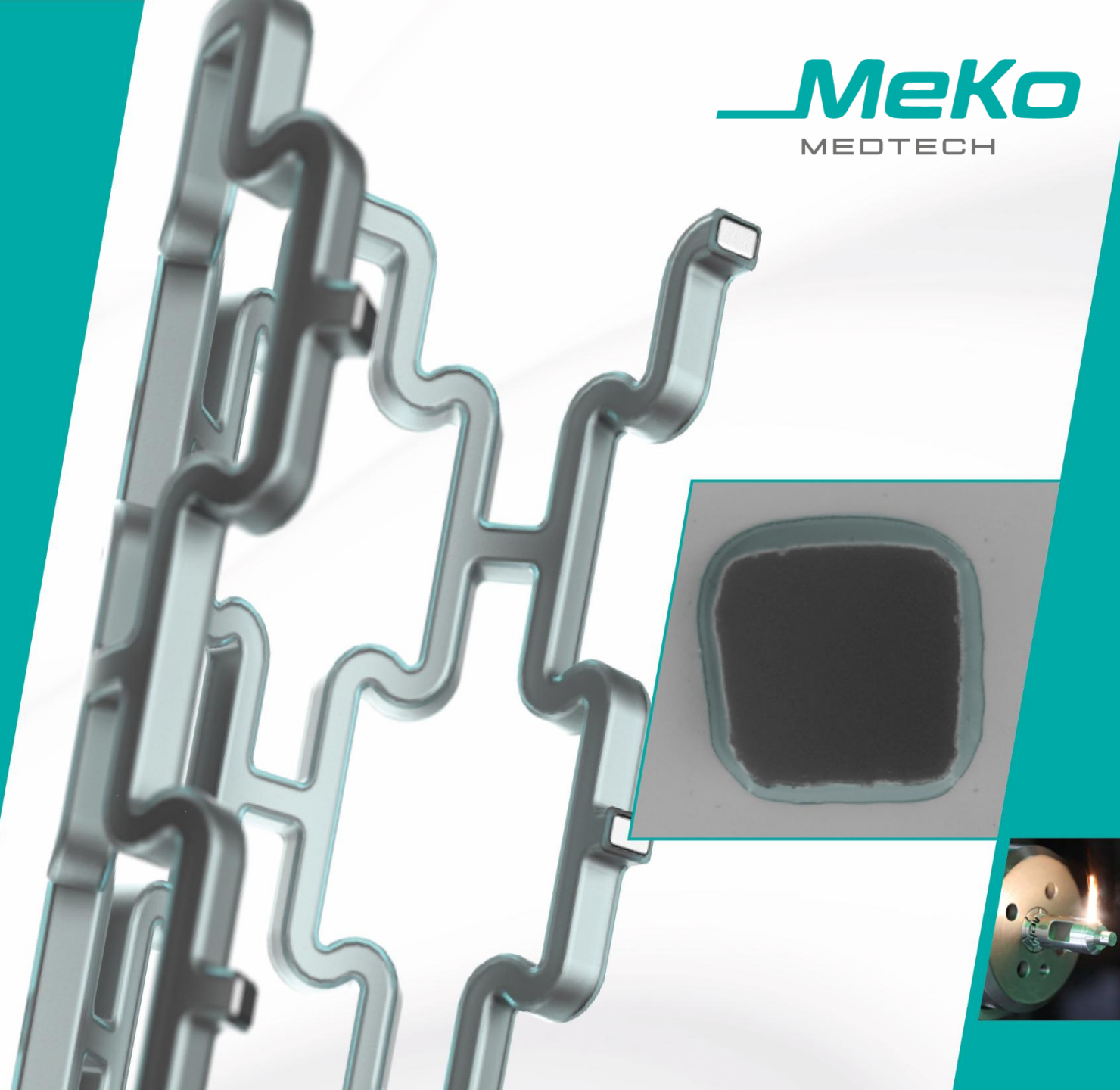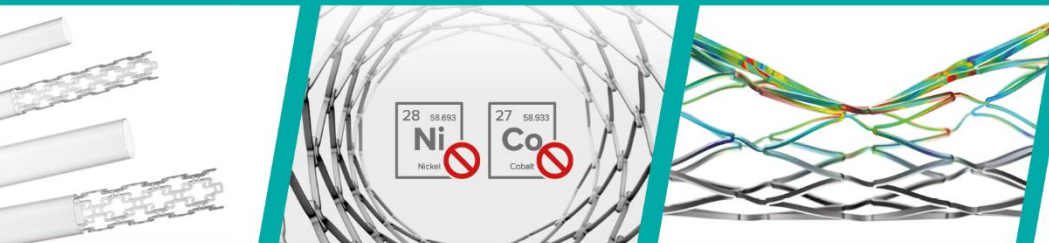


Why Coating Makes the Difference

Objective: Controlled drug release to reduce complications

- Use of Active Pharmaceutical Ingredients (API)
- Combination of stent + polymer + API → Drug Eluting Stents (DES)

Current use of DES

- DES have largely replaced Bare Metal Stent (BMS)
- Since 2023, >90% of all PCI procedures use DES
- Especially beneficial for high-risk patients and complex lesions

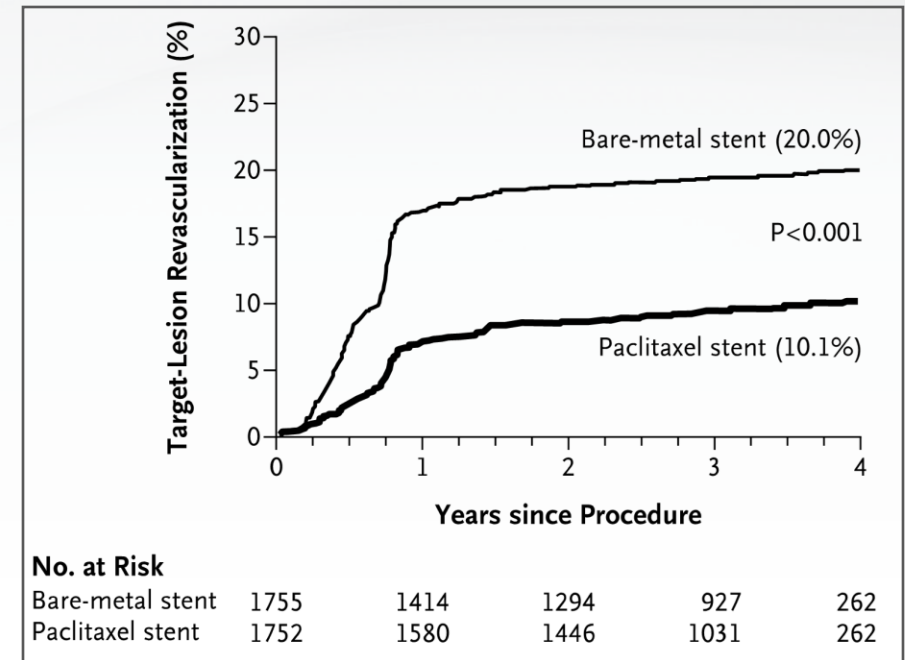

Stone, Gregg W., et al. "Safety and efficacy of sirolimus- and paclitaxel-eluting coronary stents." *New England Journal of Medicine* 356.10 (2007). doi: 10.1056/NEJMoa067193.


Our First Step into Drug-Integrated Polymers

- PolyMediX® – Development of high-precision polymer tubes with integrated Active Pharmaceutical Ingredients
- Start of our in-house expertise in combining polymers with pharmaceuticals
- Long-standing experience in polymer development, processing and functionalization
- Established analytical know-how in drug release and stability testing

PolyMediX®
by MeKo



Controlling Degradation of Resoloy®

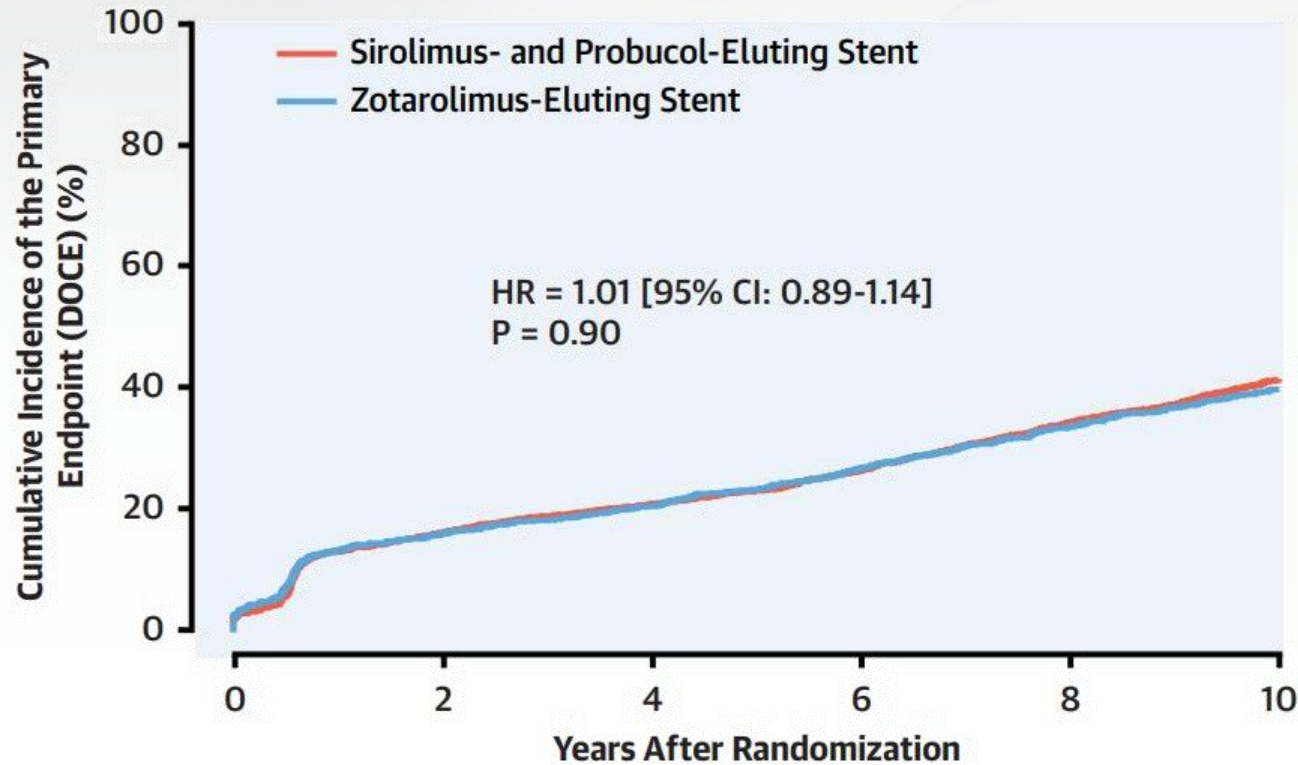


Kufner S, et al. 10-Year Outcomes From a Randomized Trial of Polymer-Free Versus Durable Polymer Drug-Eluting Coronary Stents. Journal of the American College of Cardiology. 2020; 76(2):146–158. doi:10.1016/j.jacc.2020.05.026

Controlling Degradation of Resoloy®

- Development of coatings to control the degradation of Resoloy® (bioresorbable magnesium alloy)
- In-house solution for rapid testing of various coatings

Key challenge: achieving homogeneous, reproducible coatings – beyond drug delivery

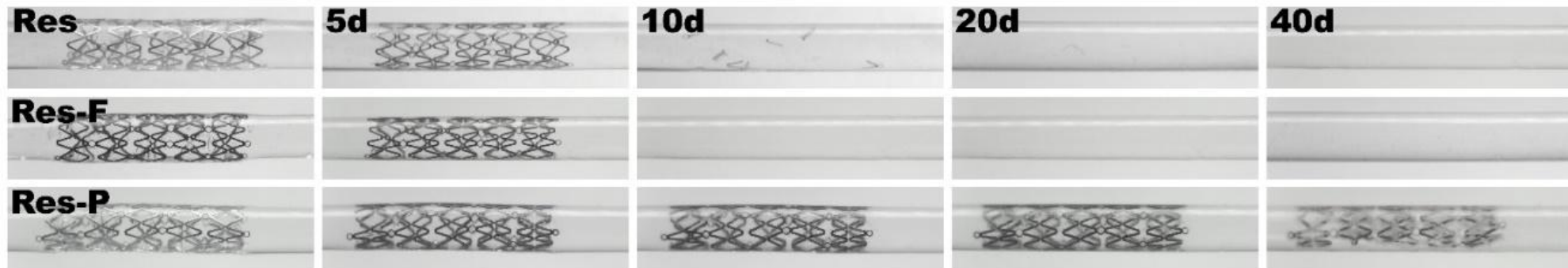

Degradation of Resoloy with different coatings

Menze R, Wittchow E. In vitro and in vivo evaluation of a novel bioresorbable magnesium scaffold with different surface modifications. 2021 Sep. doi: /10.1002/jbm.b.34790

Two Pillars of Our Coating Expertise



- Precise coatings for degradation control – beyond drug release
- Focus on homogeneity, reproducibility and quality

- Broad expertise in polymer development and processing
- Proven know-how in controlled drug release

Result: Integration of coating as a new production step at MeKo

Coating Process

Manufacturing Processes

From Solution to Coated Stent



Polymer Solution

- Formulation to customer needs

Weighing

- Precise measurement of component before coating

Precision Coating

- Tailored to customer needs

Drying

- Controlled solvent removal

Final Weighing & Control

- Verification of coating mass and thickness

API Analysis

- Optional quantification

Quality Inspection

- 100 % inspection of all implants

Shipping

- Cooled, under nitrogen atmosphere (on request)

Polymer Solutions

The First Step in Coating

Laboratory preparation

- Processing under controlled conditions (fume hoods, precision weighing)
- Dissolving polymers in suitable solvents

Polymer

- Established materials: PLGA, PLA, PLC
- Flexible testing of customer-specific polymers
- Requirement: solubility of polymer in solvent



API & Additives in Final Coating

API integration

- Tailored to customer requirements (e.g., Sirolimus, Everolimus)

API concentration in the coating depends on solution formulation and the drug itself

- Formulation – developed by MeKo or customer

Analytical control

- HPLC and DAD testing for API concentration

Additives: optional stabilizers, preservatives

Compliance: GLP-based, GMP readiness ongoing



High-Precision Coating in Cleanroom Conditions

Cleanroom environment

- Process conducted in ISO 7
- Coating stations in flow benches

Component compatibility


- Wide material choice:
Resoloy®, Vasculoy®, L605, stainless steel, Nitinol
- Cylindrical shapes, stent lengths tested: 9-45 mm
- Overly flexible parts may deform during coating

Process adaptation

- Polymer formulation tailored to material, API load and geometry
- 100% mass verification and optical inspection



Coating Process

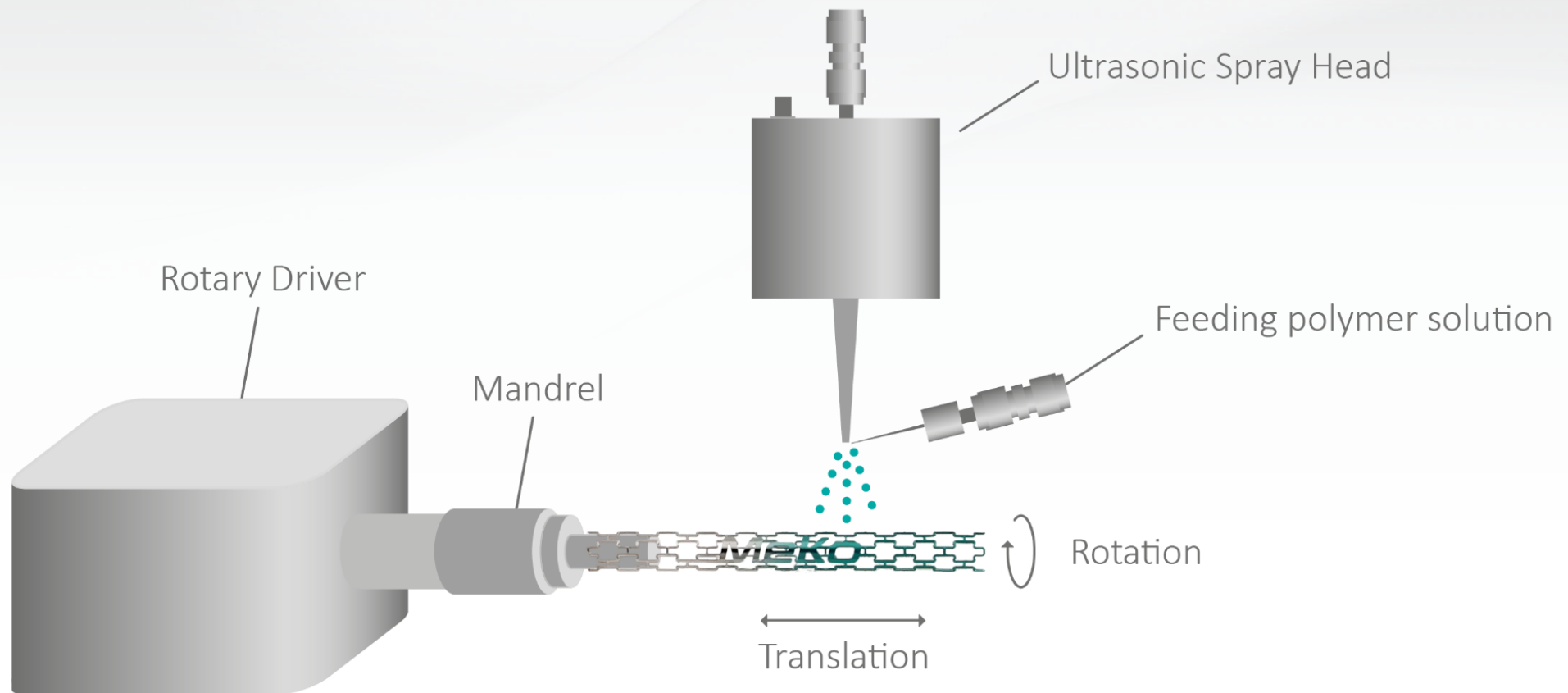

Coating Analytics

Comprehensive Coating Analytics



Mass control

- 100% verification as proxy for thickness and API load

Thickness analysis

- Point measurements or line scans (300µm) along stent length for homogeneity
- Optional cross-section analysis, material-dependent

API Analytics

- HPLC testing for drug concentration and release profile

Special analytics

- Effect on radial force
- Corrosion tests polymer and substrate behavior

Quality assurance

- 100% visual inspection of coating



High-Precision Thickness Analytics

Two-sided coating

- Overlap zone in middle, adjusted for uniform thickness

Reflection spectroscopy

- Non-contact measurement (transparent coatings only)
- Measurement range:
20 nm to 45 μm , spot size 25 μm
- System capability:
 ± 2 nm / $\pm 0.2\%$, precision up to 0.02 nm

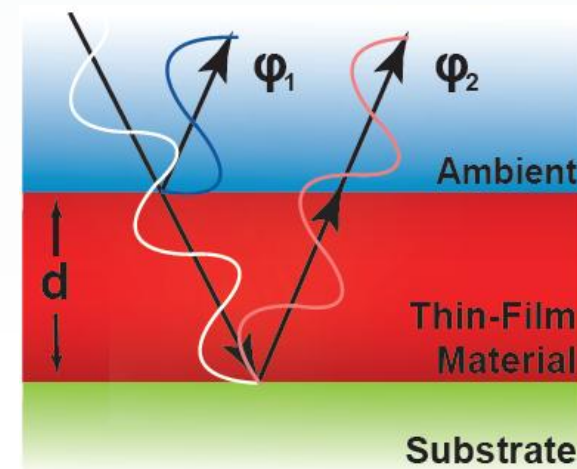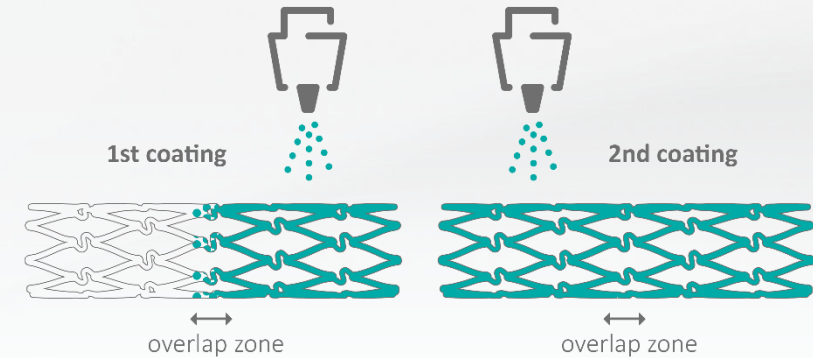

Principle of thin-film thickness measurement by reflection spectroscopy



High-Precision Thickness Analytics

Two-sided coating

- Overlap zone in middle, adjusted for uniform thickness

Reflection spectroscopy

- Non-contact measurement (transparent coatings only)
- Measurement range:
20 nm to 45 μm , spot size 25 μm
- System capability:
 ± 2 nm / $\pm 0.2\%$, precision up to 0.02 nm

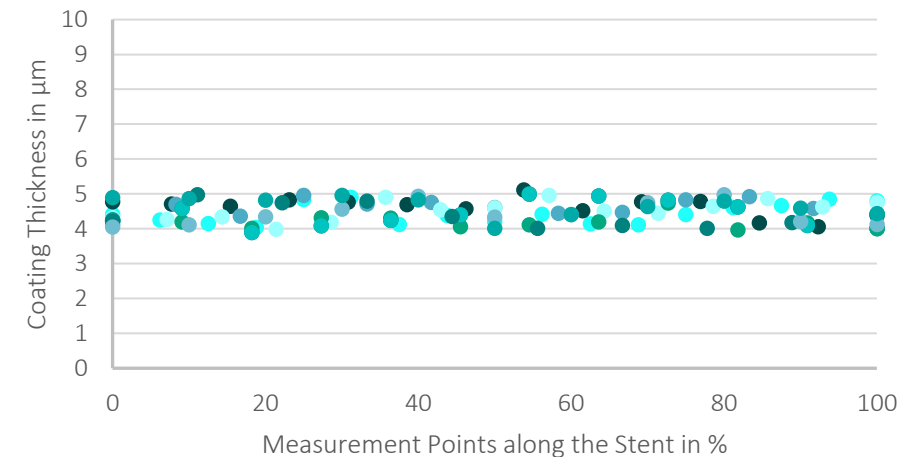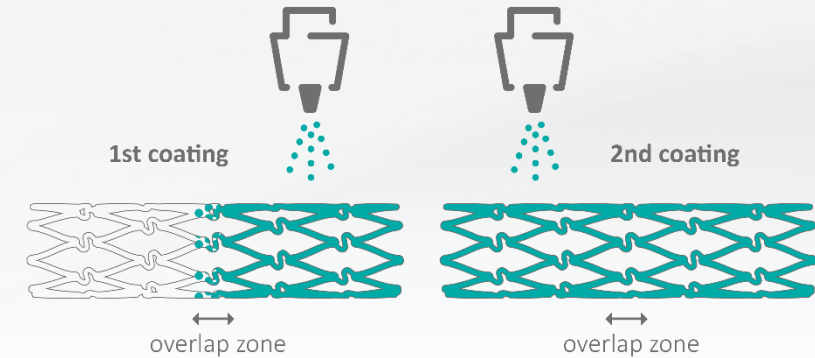


Coating thickness uniformity along stent length
– verified across 10 measured stents

Advanced Methods for Coating Thickness Analysis

Confocal Laser Scanning Microscopy (CLSM)

- Suitable for rough or complex surfaces

Coating Thickness Analysis of the abluminal layer using Confocal Laser Scanning Microscopy (CLSM)

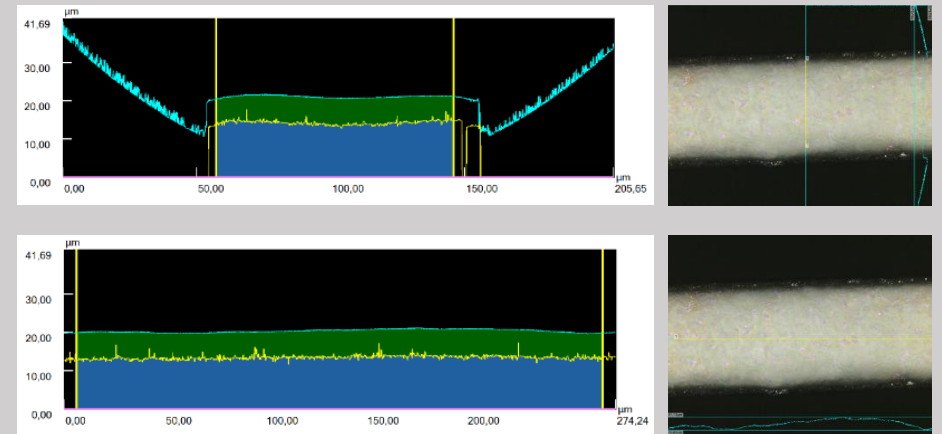

Advanced Methods for Coating Thickness Analysis

Confocal Laser Scanning Microscopy (CLSM)

- Suitable for rough or complex surfaces


Cross-Section Analysis

- Sample preparation must be feasible (material-dependent)
- Direct visualization of luminal vs. abluminal layers

Outcome:

- Both methods confirm homogeneous coatings
- Established coating thickness ranges:
 - > Resoloy® ~6-7 μm (polymer dependent)
Optimized for degradation control and drug delivery
 - > L605, Vasculoy® ~4-5 μm
Optimized for drug delivery

Cross-Section of Resoloy® Stent – Luminal vs. Abluminal Coating

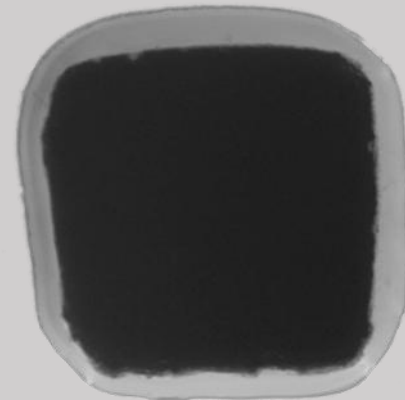

API Analytics

API Quantification & Control

Accurate quantification

- High-Performance Liquid Chromatography (HPLC) with UV/diode-array detector (DAD)

Dual control

- API content of polymer solution (before coating)
- API content of coated component (after coating)
- Typical drug loads for DES-type applications are around ~50-300 µg

Consistency

- Batch-to-batch verification of API load

Drug stability check

- Detection of degradation or impurities (e.g., SRL derivatives)



HPLC Chromatograms

API Content & Degradation

Reliable HPLC analysis


- API stability determined from peak distribution
 - > Degradation leads to extra peaks – impurities become visible
- API content determined from the peak area
 - > Ensures your coating remains consistent and effective



HPLC chromatogram of degraded sirolimus (green chromatogram) and intact sirolimus (blue chromatogram) – additional peaks indicate impurities and degradation products

API Release Testing (in-vitro)


- Reliable in-vitro testing of drug release over time
 - > Incubation setup: simple and flexible

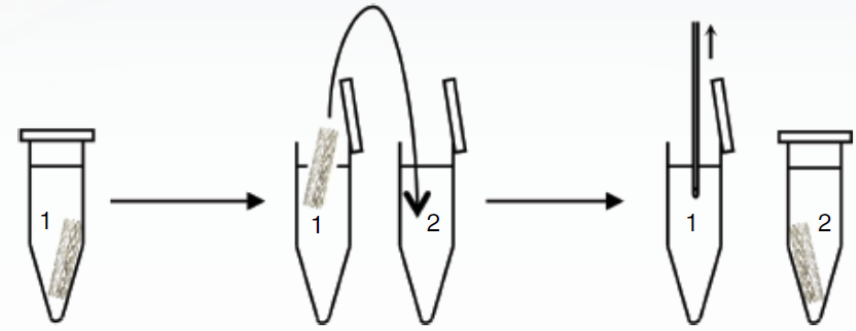

Anne Seidlitz: Drug-Eluting Stents. In: In Vitro Drug Release Testing of Special Dosage Forms (2019), S. 87–117. doi:10.1002/9781118675748

API Release Testing (in-vitro)


- Reliable in-vitro testing of drug release over time
 - > Incubation setup: simple and flexible
 - > USP 4 system (flow-through-cell): standardized, dynamic, regulatory aligned

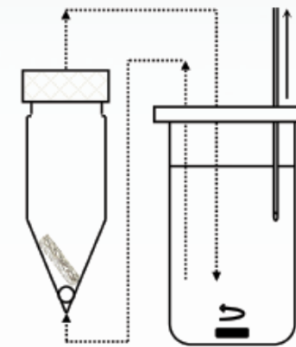

Anne Seidlitz: Drug-Eluting Stents. In: In Vitro Drug Release Testing of Special Dosage Forms (2019), S. 87–117. doi:10.1002/9781118675748


API Release Testing (in-vitro)

- Reliable in-vitro testing of drug release over time
 - > Incubation setup: simple and flexible
 - > USP 4 system (flow-through-cell): standardized, dynamic, regulatory aligned
- We help define and optimize your desired release behavior

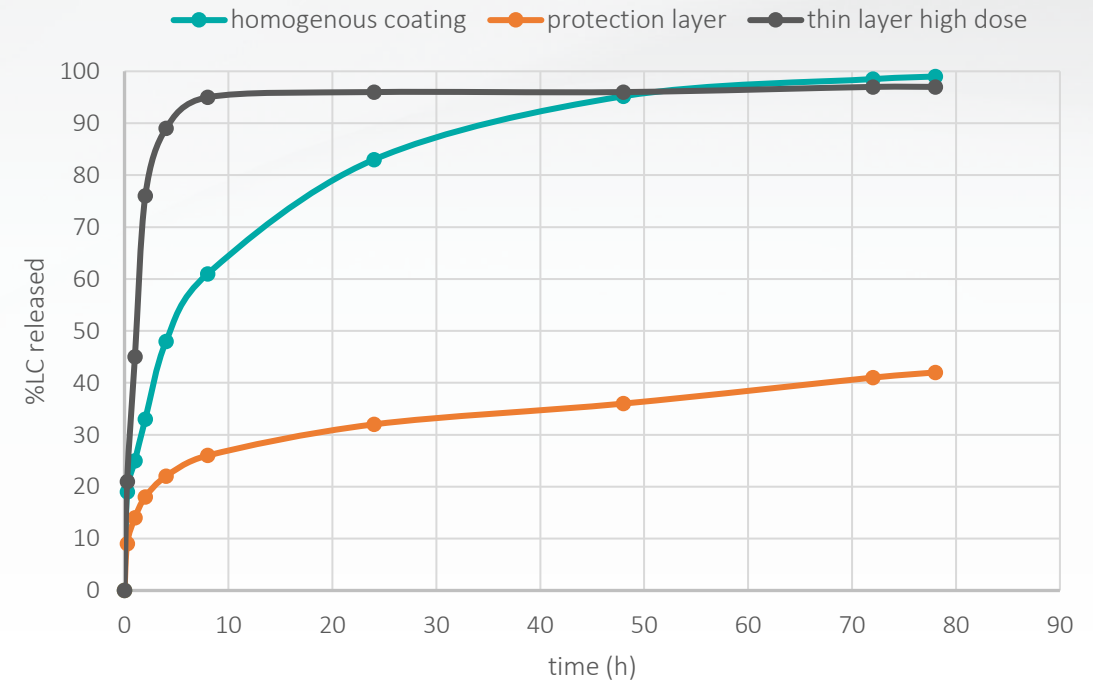

Exemplary release curve of different coating configurations
Dissolution Test, 37°C, shaking at 80 rpm

Controlled Release Influencing Factors

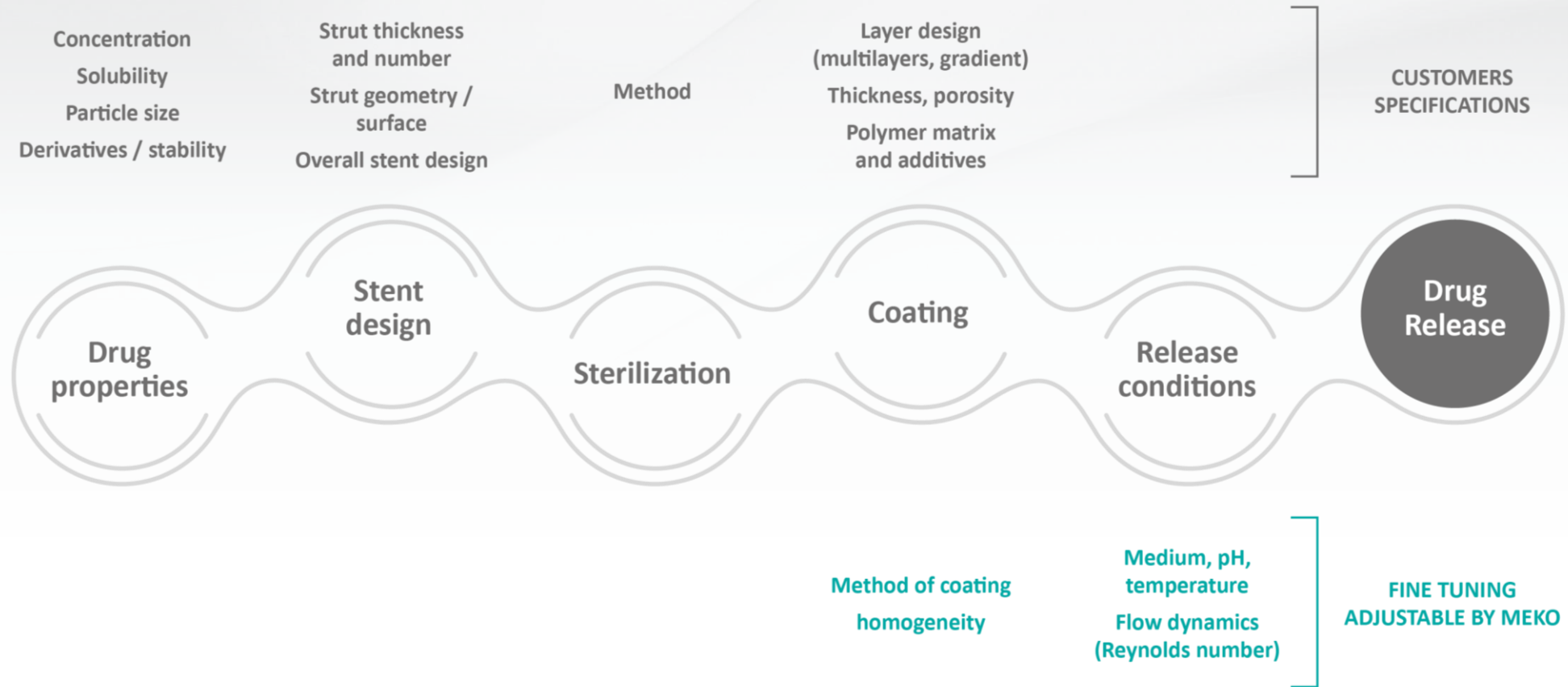

Challenges & Solutions

Coating Uniformity

Post-Stent Dilatation



Coating must withstand crimping and dilatation without delamination

Our capability

- Testing Stents under crimping/dilatation and analyzing coating behavior with SEM
- Supported by in-house FEM simulation expertise

Optimization

- Achievable through adapted parameters or thinner coating layers
- Coating performance during dilatation is strongly dependent on polymer and design

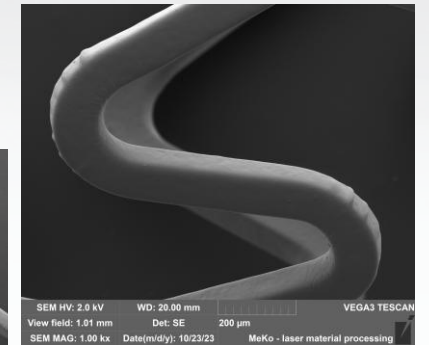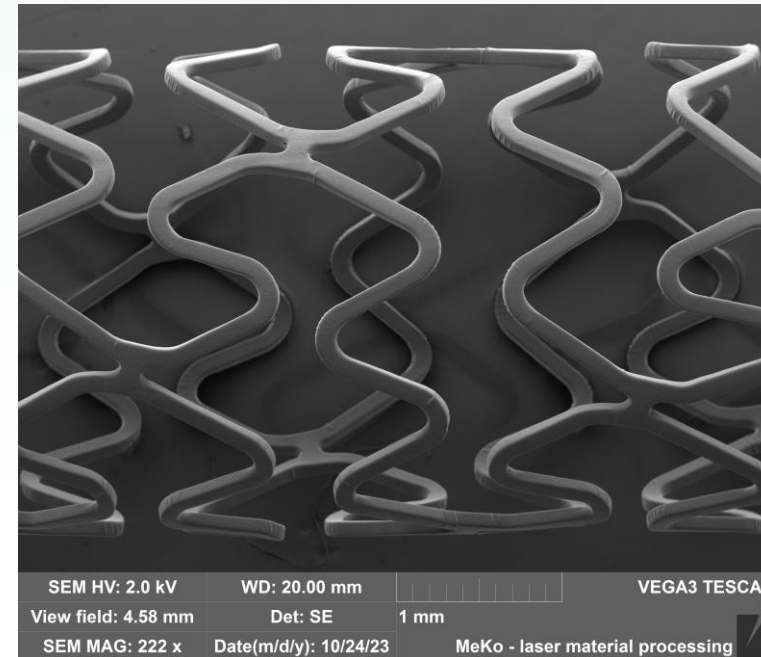

Dilated stent with a PLLA coating. With optimized settings, only tension lines without cracks or holes (SEM)

Coating Uniformity


Large-diameter Components

Challenges with components in large diameter

- For example, valve frames with OD ~15 mm
- With standard axial coating setup, cut edges initially remained uncoated

Process optimization

- Enabling uniform coating on all sides, including edges

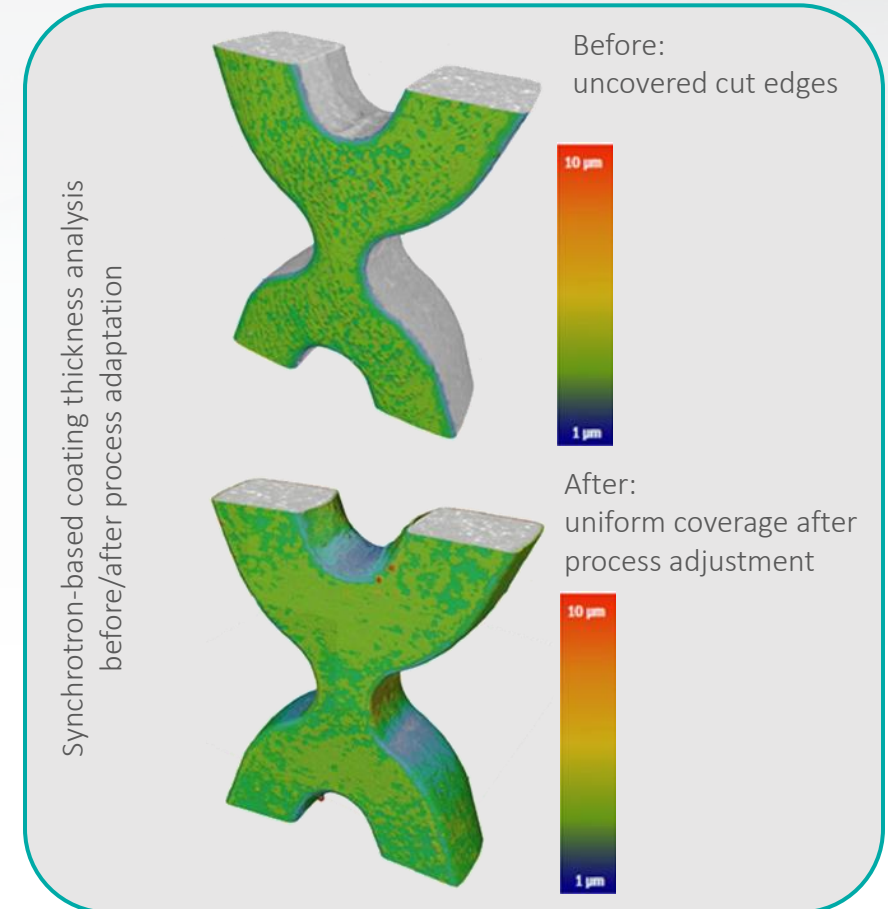

Beyond Coating

Covering as an Option

Project status

- Currently in R&D phase

Customer-driven


- Further development possible upon request

Requirements

- Tolerance specifications and component details needed

Opportunities

- Ready to expand with customer demand

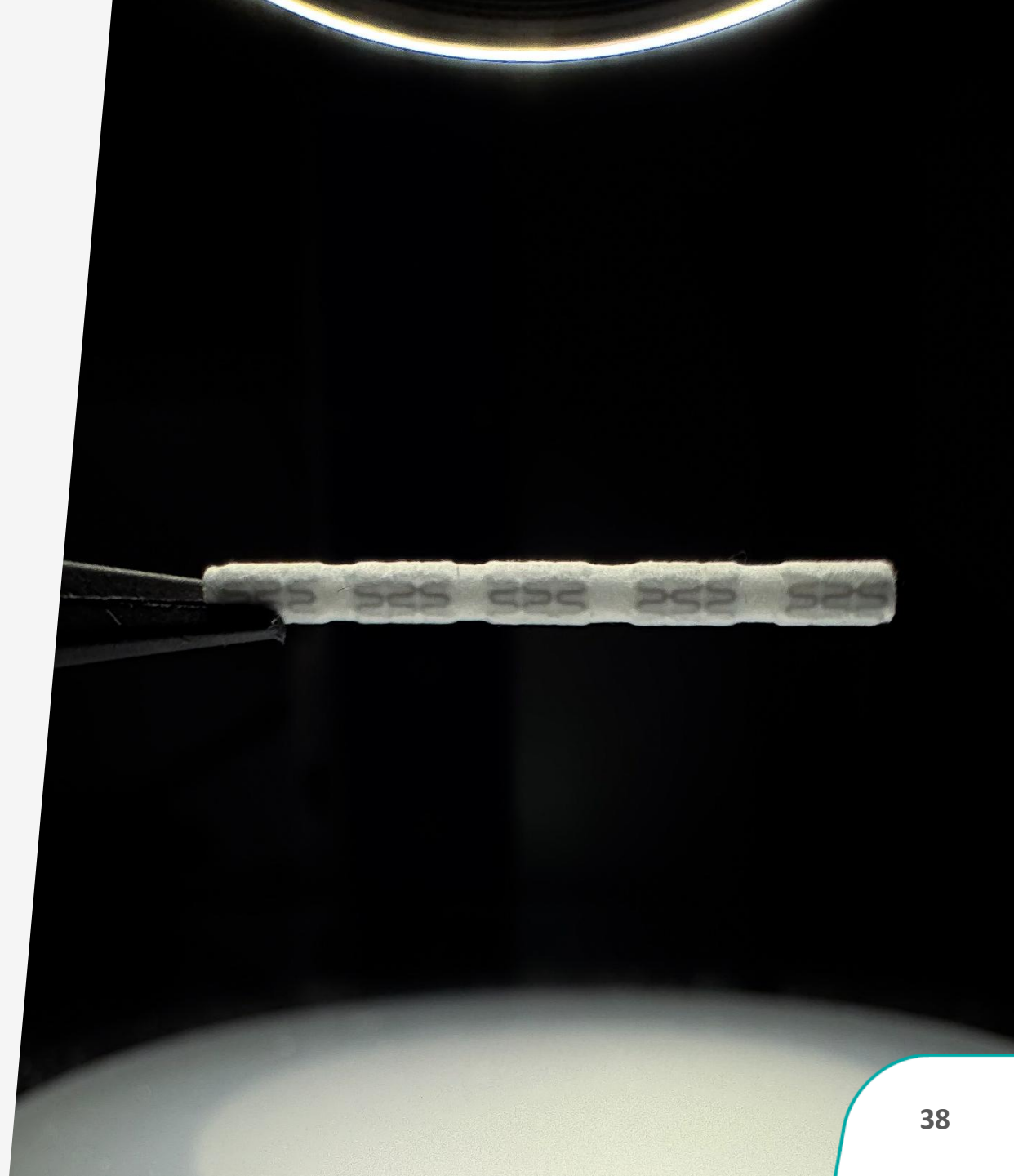

Customized Coating Solutions

Your input for a tailored coating solution:

- **Component details:** design, dimensions, tubing material
- **Polymer selection:** proven biopolymers novel candidates can be evaluated jointly
- **API specification:** drug type desired concentration (total, per length or per surface area)
- **Coating thickness:** specification for individual layers in multilayer designs

You know our capabilities

– now it's time to bring your project to life.



PRECISION IS OUR PROFESSION



MeKo Manufacturing e.K.
Sarstedt, Germany
+49 5066 7079-0
laser@meko.de

